



Orthodocs Cocoon Fitting Instructions

Orthodocs Limited
9 Mallard View
Newtownards
BT23 4FB

08000 480 270
www.orthodocs.co.uk



1. Remove Cocoon from packaging & place on the floor in front of the bed



2. Lower the siderails nearest to you and remove the mattress.



3. Position the Cocoon on the bed as shown.



4. Position the end bumpers against the head/footboards as shown.



5. Position the side bumper against the far siderails & attach loosely using the velcro straps.



6. Zip together the side and end bumpers at the far side.




7. Lower the side bumper closest to you to reveal the mattress 'envelope'.




8. ** Position the mattress within the 'envelope' - easing it under the bumpers.

****TIP**
It is much easier to fit sheets to the mattress *before* attempting Step 8.

DYNAMIC MATTRESS
A dynamic mattress can be used with Cocoon. Fitting is similar to a standard mattress but you should ensure that the air pipes are fed through one of the holes in the lower surface of the Cocoon.

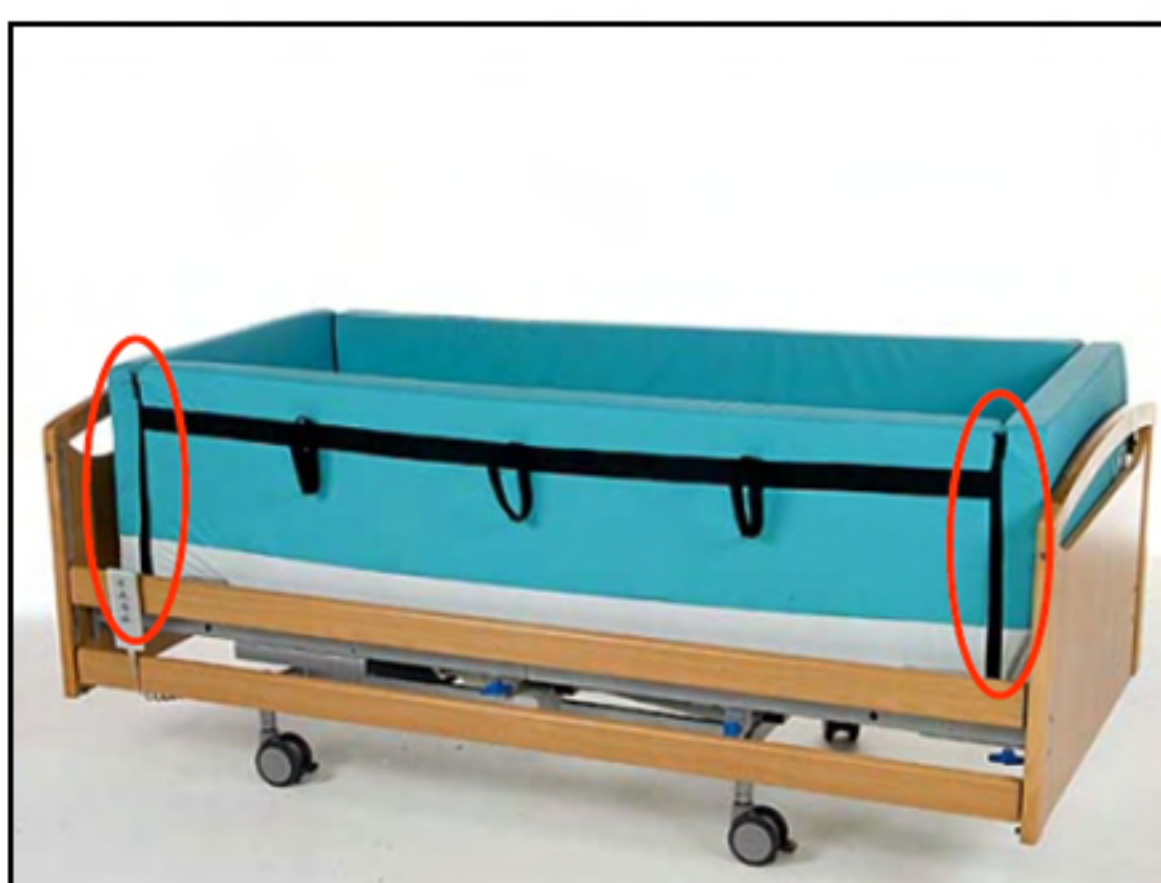

9. Raise the side bumper into position & zip up both end bumpers.

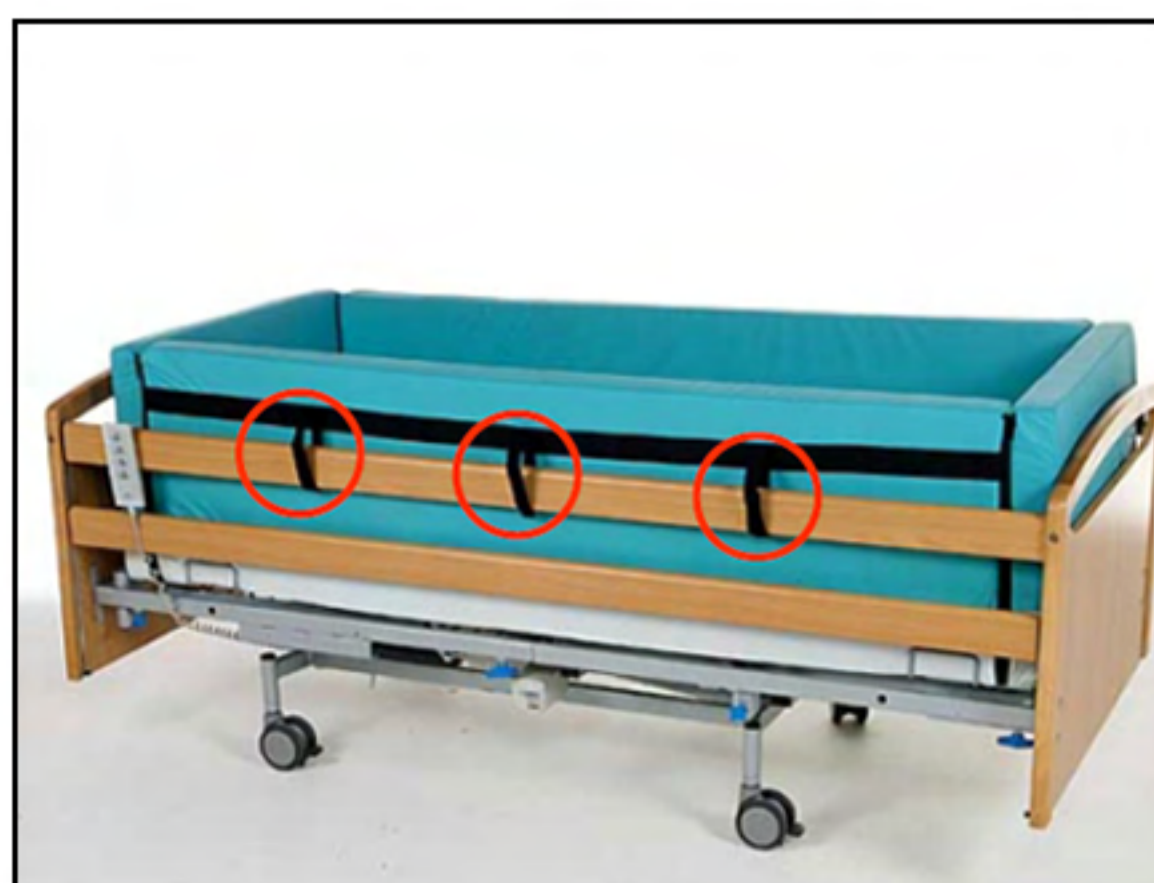

10. Raise the siderails and attach the Cocoon using the velcro straps.



NOTE : Slight Profiling is possible if the Velcro Straps are released as shown - Otherwise please discuss "Profiling Solutions" with us